

Note:

We recommend trying out Waypi® Man at your leisure before using it for the first time. The use of a urinating funnel may be unfamiliar at first. The Waypi® Man urinating funnel is available in pharmacies or from specialist retailers.



New!

Waypi[®] Man

Urinating funnel for man

Material: 100 % TPE 

EAN 4013273002127

Made in Germany

Manufacturer:

KESSEL medintim GmbH
Nordendstr. 82 – 84
64546 Mörfelden-Walldorf
Germany

Phone: +49 6105 20 37 20

www.medintim.de

14.04.2021

Medintim

Some men do not have an easy time passing urine: With physical limitations, after an operation or if they are overweight, men can urinate comfortably and cleanly while standing or sitting with the Waypi® Man urinating funnel – hygienic, simple, discreet and practical.

reusable,
ecological,
no unnecessary
waste

How to use the Waypi® Man correctly:

Take the still folded Waypi® Man out of its storage box. To unfold, simply lift the loop and the Waypi® Man unfolds by itself.

To use, hold the Waypi® Man urinating funnel over the penis, with the loop pointing the body. The end of the funnel points diagonally forward towards the ground. Make sure that the Waypi® Man is covering the urethral outlet.

After use, clean the Waypi® Man with soap and water, fold it and store it in the storage box. It has a small ventilation slot on the underside to avoid moisture collecting in the box.

Advantages of the Waypi® Man:

- extra smooth, satin finished surface
- easy to clean
- conforms to body contours
- innovative shape prevents overflow and splashing
- with a loop for practical folding
- with discrete storage box
- also available with drain hose on request
- fits all bottles if urine is to be collected

